

Medipoint® 26 Area Alarm

The BeaconMedæS Medipoint® 26 Medical Gas Area Alarm is designed to monitor up to six medical gas services in the Normal, High and Low pressure conditions. They are normally used to provide a warning of abnormal operating pressure downstream of departmental area valve service units. Line contact monitoring circuits constantly verify the integrity of pressure switch and interconnecting wiring circuits.

Antimicrobial Fascia

The fascia is made from Autotex AM. Microban® technology is incorporated into the Autotex textured hardcoat during the manufacturing process. When bacteria come into contact with Autotex AM with Microban®, the antimicrobial function disrupts the bacterial cell wall, killing or inhibiting bacterial growth.



Autotex AM has been tested to AATCC Test Method 100⁷, and exhibited biocidal passes against a wide range of microbes including MRSA and E. coli.

Surface Mount Technology

The Medipoint 26 now comes with advanced printed circuit boards using surface mount technology as standard. This allows cooler operating temperatures to be maintained, increasing the reliability and substantially lengthening service life.

*Microban and the Microban symbol are trademarks of the Microban Products Company, Huntersville, NC, USA

*Life
is in the
details.*



High Specification Switches


High specification, virtually drift-free pressure switches include dual double throw contacts to monitor both high and low pressure in a single switch. A miniature PCB installed in the switch cover includes the line contact monitoring circuit so that no additional junction boxes are needed.

Simple Reprogramming

The Medipoint® 26 Alarm is easy to reprogramme on site if modifications are made to your medical gas system. A simple 12-way switch can turn additional columns on so that a more gas services can be monitored - no mess, no fuss - simple.

Adjustable Volume Control

The alarm also includes a volume control so that the volume can be set to suit the area of installation. This feature is of great benefit in areas where alarms are positioned close to monitored nurse bases where patients should not be disturbed.

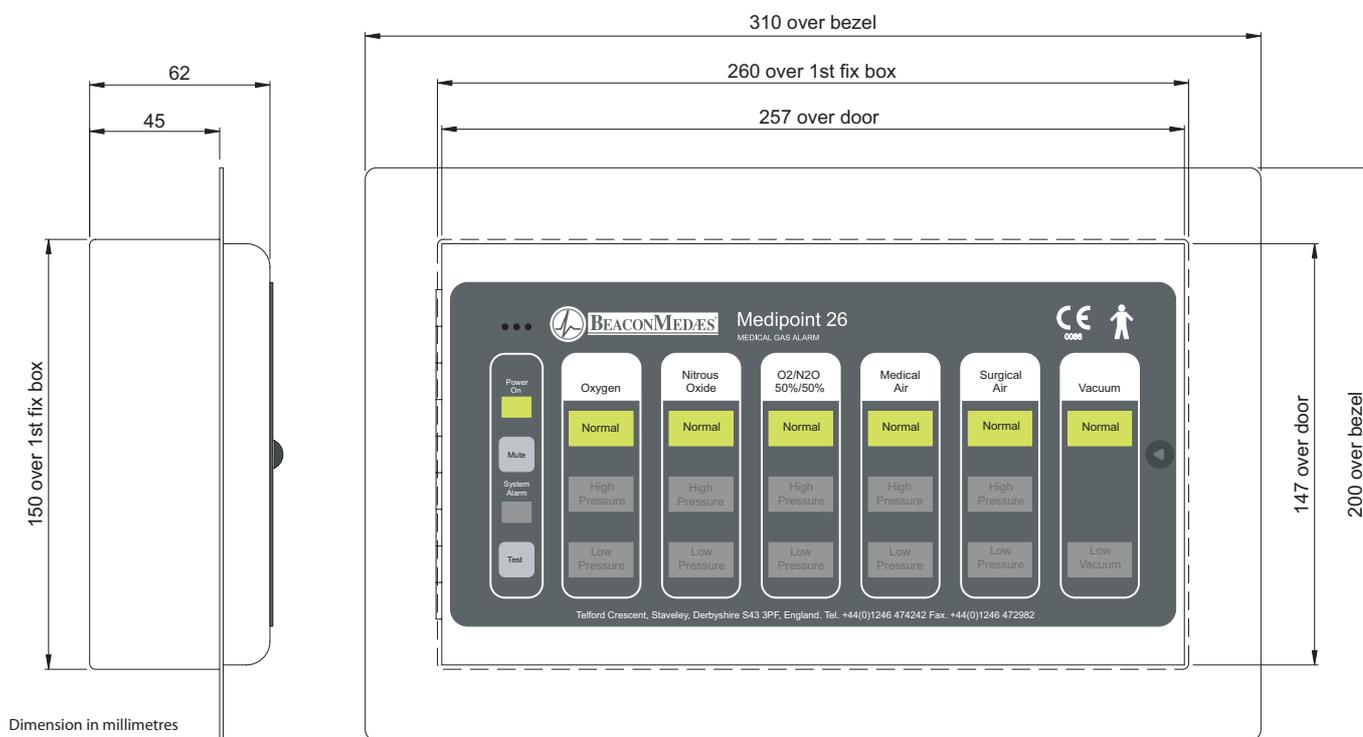

Medipoint® 26 Area Alarm

Description	Part Number	Description	Part Number
1st fix box	1826481	Pressure switch tee - 15mm	1824433
Bezel for flush installations	1826484	Pressure switch tee - 22mm	1824434
2nd fix for 1-6 gases - 230V	1828815	Pressure switch tee - 28mm	1826284
2nd fix for 1-6 gases - 110V	1828821	Pressure switch tee - 35mm	1826285
Pressure switch 400 kPa	1829936	Pressure switch tee - 42mm	1826286
Pressure switch 700 kPa	1829937	Pressure switch tee - 54mm	1826287
Vacuum switch -40 kPa	1829938	Pressure switch connection kit	1829939
Remote audible	1826610	3 core data cable	1828809

BeaconMedaEs
A company within the Atlas Copco Group
Telford Crescent
Staveley, Derbyshire
S43 3PF, England

Tel: +44 (0) 1246 474 242
Fax: +44 (0) 1246 472 982
Email: gbn.info@beaconmedaes.com
Web: www.medaes.com

Medipoint® 26 Area Alarm

